



Atex Co.,Ltd.

144 , Jangjisan-ro, Gwangtan-myeon, Paju-si, Gyeonggi-do,
10955, Korea

Tel : +82 31 948 6654 Fax +82 31 942 6182

Email : atexsales@atexmedical.com

Technical Datasheet

<i>Description</i>	Mono-extensible adhesive elastic bandage in cotton fabric (97%) and elastan or spandex (3%) The fabric is coated with the acrylic adhesive mass so as to form waves. The adhesive side of the bandage is protected by an easily removable paper.
<i>Details</i>	It is used to make bandages acting on body muscles and joints.
<i>Intended use</i>	Dream®K should be applied to the patient and used exclusively by qualified professionals in the health sector (doctors, physiotherapists, sports masseurs). It is suitable for proprioceptive taping
<i>Support</i>	Cotone 97%; Elastan 3%
<i>Weight of yarn</i>	approx. 170 g/cm ²
<i>Adhesive mass</i>	Acrylic adhesive mass composed of esterified copolymers (acrylic polymers, ethyl acetate, solid particles).
<i>Elasticity</i>	140% elongation
<i>Height</i>	2,5cm ± 2% 5cm ± 2% 7,5cm ± 2%
<i>Length</i>	At rest: 5 m (± 2%) – 32 m (± 2%)
<i>Colours</i>	Due to commercial reasons, the device is available in 9 different colours: yellow, beige, fuchsia pink, red, blue, black, tribe blue/light blue, tribe white/blue, tribe white/black, tribe black/light blue, tribe yellow/red, tribe red/black, tribe white/green, tribe white/purple, tribe silver, tribe orange/white, tribe yellow/blue, tribe red/blue, tribe red white.rds
<i>How to use</i>	Before applying Dream® K , clean the skin with soap and water, then dry accurately. In case thick body hairs are present, shave the part before applying the tape. Body temperature helps to increase the adherence, so it is recommended to apply Dream® K at least 30-60 minutes before physical activity. To remove the tape, according to the skin structure, hold firmly the skin with one hand while carefully and steadily remove Dream® K with the other hand (avoid sudden



Atex Co.,Ltd.

144 , Jangjisan-ro, Gwangtan-myeon, Paju-si, Gyeonggi-do,
10955, Korea

Tel : +82 31 948 6654 Fax +82 31 942 6182

Email : atexsales@atexmedical.com

	movements).
<i>Pros</i>	It does not contain latex.
<i>Warnings</i>	Dream® K must be applied to the patient and used only by qualified healthcare professionals (i.e. Doctors, Physiotherapists, Sport Masseurs). Do not apply to red or irritated skin or on wounds. After applying the tape, if redness, irritation or itching of the skin occur, remove it immediately. Although the product is dermatologically tested, if the skin is known to be particularly delicate and sensitive, or if there is a predisposition for allergies to adhesives, certain types of fabric or colours, it is recommended to test a few centimeters of the product on the skin before application. Dream® K is affected by humidity. Store in a cool dry place away from direct heat. Keep out of the reach of children. Do not use if packaging is damaged.
<i>Packaging</i>	Any single bandage is supplied already wrapped in cellophane, with all the necessary information for a simple identification and sold in the smallest piece of packaging: - 12 pcs. (Size 2.5 cm) - 6 pcs (size 5 cm) - 4 pcs (size 7.5 cm) - 1 pcs (size 5 cm)
<i>Use and sterilization</i>	Single-use, non-sterile, non-invasive
<i>Storage and lifetime</i>	The ideal temperature for the product to be stored is < 30 °c The product lifetime is 3 years from the date of production The device fears humidity Store in a cool, dry place, away from direct sources of heat Do not use if the packaging appears to be damaged
<i>Disposal</i>	Please, follow the relevant laws and/or regional provisions
<i>Applicable rules & Standard applied</i>	Class I based on Rule 1, Annex VIII, Regulation (EU) 2017/745
<i>Distributor / Importer</i>	Sixtus Italia srl Via Tourcoing, 23 59100 Prato (PO) Italy Tel. 0574.7561 – Fax 0574.756211
<i>EC REP</i>	Meridius Medical Europe Ltd. Unit 3D, North Point House, North Point Business Park, New Mallow Road, Cork, T23 AT2P, Ireland



Atex Co.,Ltd.

144 , Jangjisan-ro, Gwangtan-myeon, Paju-si, Gyeonggi-do,
10955, Korea

Tel : +82 31 948 6654 Fax +82 31 942 6182

Email : atexsales@atexmedical.com

Version	Code
Yellow cm 2,5 x 5 m x 12 pcs	DRM0020
Light blue cm 2,5 x 5 m x 12 pcs	DRM0024
Beige cm 2,5 x 5 m x 12 pcs	DRM0022
Yellow cm 5 x 5 m x 6 pcs.	DRM0026
Beige cm 5 x 5 m x 6 pcs	DRM0028
Pink cm 5 x 5 mx 6 pcs	DRM0027
Blue cm 5 x 5 m x 6 pcs	DRM0030
Black cm 5 x 5 m x 6 pcs	DRM0031
Red cm 5 x 5 x 6 pcs	DRM0029
Yellow cm 7,5 x 5 m x 4 pcs	DRM0032
Blue cm 7,5 x 5 m x 4 pcs	DRM0036
Beige cm 7,5 x 5 m x 4 pcs	DRM0034
Black cm 7,5 x 5 m x 4 pcs	DRM0037
Tribe light blue/blue cm 5 x 5 m x 6 pcs	DRM0100
Tribe white/blue cm 5 x 5 m x 6 pcs	DRM0101
Tribe black/light blue cm 5 x 5 m x 6 pcs	DRM0102
Tribe yellow/red cm 5 x 5 m x 6 pcs	DRM0103
Tribe red/black cm 5 x 5 m x 6 pcs	DRM0104
Tribe orange/white cm 5 x 5 m x 6 pcs	DRM0105
Tribe white/black cm 5 x 5 m x 6 pcs	DRM0106
Tribe white/green cm 5 x 5 m x 6 pcs	DRM0107
Tribe white/purple cm 5 x 5 m x 6 pcs	DRM0108
Tribe yellow/blue cm 5 x 5 m x 6 pcs	DRM0109
Tribe red/white cm 5 x 5 m x 6 pcs	DRM0110
Tribe silver cm 5 x 5 m x 6pcs	DRM0111
Tribe red/blue cm 5 x 5 m x 6 pcs	DRM0112
Yellow cm 5 x 32 m	DRM0026/L
Beige cm 5 x 32 m.	DRM0028/L
Pink cm 5 x 32 m.	DRM0027/L
Blue cm 5 x 32 m.	DRM0030/L
Black cm 5 x 32 m.	DRM0031/L
Red cm 5 x 32 m.	DRM0029/L
Yellow cm 5X5 m	DRM0026/S
Pink cm 5 X 5 m	DRM0027/S
Beige cm 5 X 5 m	DRM0028/S
Red cm 5 X 5 m	DRM0029/S
Blue cm 5 X 5 m	DRM0030/S
Black cm 5 X 5 m	DRM0031/S
Blue/light blue cm 5 X 5 m	DRM0100/S
White/blue cm 5 X 5 m	DRM0101/S
Black/blue cm 5 X 5 m	DRM0102/S
Yellow/red cm 5 X 5 m	DRM0103/S



Atex Co.,Ltd.

144 , Jangjisan-ro, Gwangtan-myeon, Paju-si, Gyeonggi-do,
10955, Korea

Tel : +82 31 948 6654 Fax +82 31 942 6182

Email : atexsales@atexmedical.com

Red/black cm 5 X 5 m	DRM0104/S
White/orange cm 5 X 5 m	DRM0105/S
White/black cm 5 X 5 m	DRM0106/S
White/green cm 5 X 5 m	DRM0107/S
White/purple cm 5 X 5 m	DRM0108/S
Yellow/blue cm 5 X 5 m	DRM0109/S
White/red cm 5 X 5 m	DRM0110/S
Red/blue cm 5 X 5 m	DRM0112/S