Rev. 2 - 05/09_100149

Bauerfeind AG
Weissendorfer Strasse 5
07937 Zeulenroda-Triebes
Germany
Phone +49 (0) 36628-66-40 00
Fax +49 (0) 36628-66-44 99
E-mail info@bauerfeind.com

GenuTrain® S

Aktivbandage
Active support
Orthèse active
Vendaje activo

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GB Dear customer,

thank you for choosing a product from Bauerfeind. GenuTrain® S is a high-quality medical product.

Please read through these instructions for use carefully and if you have any questions contact your medical provider, medical retailer or our Technical Service department.

GenuTrain® S, the active knee support, produces its effect primarily during movement. The anatomically-contoured lateral hinges are held directly by the Velcro® strap system, thereby supporting the natural sequence of movements. During movement, the shaped viscoelastic ring compresses and massages the soft tissues. The enhanced circulation in the joint capsule and the promotion of lymph drainage lead to reduced swelling, pain relief and improved function while, at the same time, taking the strain off the kneecap.

The three-dimensional knitting technique ensures a perfect fit for GenuTrain® S. Graduated compression at the ends of the support prevents it from rolling and cutting into the leg.

GenuTrain® S must be fitted for the first use by a trained specialist, otherwise we cannot accept any liability for the product.

Indications

- Mild instability
- Inflammatory changes of the knee (arthritis) e.g. rheumatoid arthritis
- Worn knee joint

Side effects

Side effects involving the body as a whole have not been reported to date. Correct fitting is assumed. Any supports and orthoses¹ applied externally to the body can, if tightened excessively, lead to local pressure signs or, rarely, constrict the underlying blood vessels or nerves.

Contra-indications

Hypersensitive reactions harmful to health have not been reported to date. In the following conditions such aids should only be fitted and applied after consultation with your medical provider:

1. Skin disorders/injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.
2. Varicose veins.
3. Impaired sensation and circulatory disorders of the legs, e.g. diabetes mellitus.
4. Impaired lymph drainage – including soft tissue swellings of uncertain origin located remotely from the fitted aid.

Fitting GenuTrain® S/Notes for the orthopaedic technician

- Check the fit of the hinges against the patient's leg, particularly in relation to specific anatomical features (e.g. the top of the calfbone), (Figure 1).
- Insert the slightly flexed hinges into the hinge pocket from the proximal (top) end as per the markings (Figure 2).
- To fully insert the hinges, extend the hinge pocket by pulling on the draw loops and the lower hinges (Figure 3).
- Insert the hinges on the inner and outer aspects of the knee in the same way.
- The velcro® straps should preferably be fastened at the hinge on the inside of the knee (Figure 4).
- The wider straps are intended for the thigh area and the narrow straps for the lower leg. The shorter straps are used at the front in each case and the longer straps at the back (Figure 5).
- Before fitting the support, pull all four straps through the outer strap loops on the hinges and loosely secure the Velcro® fastening at the maximum width (Figure 6).
- To fit the support, grasp the lateral draw loops and slip the support over the knee so that the kneecap is surrounded by the shaped ring without exerting any pressure (Figure 7/8).
- After rechecking the position, uniformly tighten and fasten the straps to achieve a correct fit (Figure 9).
- The Velcro® strap tension should be checked after 10-15 minutes to avoid painful compression and congestion and adapted if necessary.

- To remove GenuTrain® S, grasp the support from below and pull off the leg (Figure 10).
- The hinges are removed in the reverse sequence. The straps are completely removed, and the hinges are withdrawn through the lower opening of the hinge pocket at the back (Figure 11).

Notes for the orthopaedic technician

If required, the hinges of GenuTrain® S can be reshaped with heating at the appropriate points in order to ensure a perfect fit. To this end, the material is carefully heated on both sides with a hot-air gun to approx. 120-140°C (248-284°F). Ensure that the Velcro® straps are removed beforehand. Always observe the safety directions when working with heat-generating equipment.

Important notes

- GenuTrain® S is a prescribed product that should be applied under a physician's guidance. GenuTrain® S should only be applied in accordance with the instructions contained in these usage directions and for the listed areas of application.
- The product is intended to be fitted only once and to one patient.
- No liability is accepted for improper use of the product.
- Improper changes must not be made for the product. Non compliance can reduce the performance of the product and thus no liability will be accepted.
- Should you notice any unusual changes (e.g. increased symptoms), please contact your medical provider without delay.
- A combination with other products (e.g. compression stockings) must be discussed beforehand with your medical provider.
- The product's unique action is produced primarily during physical activity. The product should be removed during prolonged rest breaks (e.g. sleeping, continuous sitting).
- Avoid the use of ointments, lotions or any substances including grease or acids.
- Guarantee subject to statutory regulations.
- The product can be disposed of without any restrictions.

Washing instructions and product care

All Train products² can be machine washed separately at 30 °C with liquid detergent. Please note the following when washing your Train products:

1. For products with Velcro® straps: Remove the straps if possible, or fasten them securely at the place provided for this purpose.
2. If possible, also remove pads/inserts and their terrycloth covers before washing. We recommend using a mesh bag. Air dry your Train product. Do not dry it in a dryer because this can compromise the knitted fabric.

Please wash your Train product regularly so that the compression of the knitted fabric can be retained.

This product has been examined by our own quality control system. However, should you have any complaint, please contact your retailer or our customer service department.

Version: May 2019

¹ Orthese = orthopaedic appliance used to stabilize, relieve, immobilize, control or correct the limbs or the torso

² Standardized washing tests show that after 50 washing machine cycles full functionality of the Train® active support is retained. (Liquid detergent was used.)

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