Rev. 2 - 07/09_103641

Material
Polypropylen (PP)
Etylen-CO-Polymere
Polyamid (PA)
Acrylat Klebstoff
Siebdruckfarbe/ Siebdruckkleber
Cyanacrylat

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Dear client,

thank you for choosing a product from Bauerfeind. MalleoLoc® is a high-quality medical product.

Please read through these instructions for use carefully and if you have any questions contact your doctor, medical retailer or our Technical Service department.

MalleoLoc® is an anatomically contoured stabilizing orthosis¹ for the ankle. MalleoLoc® effectively counteracts lateral ligament instability while promoting natural muscular stabilization. The adjustable Velcro® straps and thermoplastic material allow MalleoLoc® to be individually fitted. MalleoLoc® fits into almost any shoe for street or sport wear.

⚠ MalleoLoc® must be individually adapted to the patient by a trained orthopaedic technician in order to gain the maximum effect and optimal wear comfort from this orthosis.

Indications

- Early functional treatment for injuries of the lateral malleolar ligaments (and the bifurcate ligaments)
- Capsular ligament strains
- Postoperative rehabilitation
- Chronic ligament insufficiency
- Prophylactic use

Side effects

Side effects, that affect the entire organism, have not been reported to date. Correct fitting is assumed. Any supports and orthoses applied externally to the body can, if tightened excessively, lead to local pressure signs or, rarely, constrict the underlying blood vessels or nerves.

Contra-indications

Hypersensitive reactions harmful to health have not been reported to date. In the following conditions such aids should only be fitted and applied after consultation with your medical provider:

1. Skin disorders/injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.
2. Varicose veins.
3. Impaired sensation and circulatory disorders of the legs/feet, e. g. Diabetes mellitus.
4. Impaired lymph drainage – including soft tissue swellings of uncertain origin located remotely from the fitted aid.

Fitting MalleoLoc®

1. Open Velcro® straps A and B, removing them from the eyelets and detaching them from the corresponding Velcro® points. Loosen strap C.
2. Position MalleoLoc® so that the U-shaped stirrup fits under the arch of the foot, leaving the heel free. The sides of the brace should lie laterally (outside) in front of the malleolus and medially (inside) in back of the malleolus (Fig. 1). When fitting the stabilizing support, ensure that the heel remains in contact with the ground. Secure strap C with the Velcro® closure and ensure that the padding rests against the Achilles tendon and that MalleoLoc® remains in position around the foot.
3. Pass strap B around the lower leg, keeping it flat against the leg, and secure the top Velcro® point. Then pass the strap through the opposite eyelet securing the end of the strap with the Velcro® closure (Fig. 2).
4. Secure strap A by applying the light blue portion to the light blue Velcro® point and the grey portion to the grey Velcro® point. Ensuring that the strap lies flat against the leg. Pull the end of the Velcro® strap A through the respective eyelet. Secure using the Velcro® closure (Fig. 3).
5. If necessary, correct the position of the Velcro® straps (Fig. 4). The separate cushion enclosed may be attached to strap A (e. g. in front of the foot or in the Achilles tendon area) if necessary.

⚠ Important!

All three Velcro® straps can be individually adjusted at both ends.

⚠ Instructions for the Orthopaedic Technician

If the patient's anatomy requires or in order to allow for extreme athletic activity, the plastic material of the MalleoLoc® must be shaped by heating it at targeted points to ensure a perfect fit. Carefully remove the blue inner liner before heating the thermoplastic material. Then carefully heat the material on both sides to approx. 200°C (392°F) using a heat gun, ensuring that the Velcro® fastenings remain covered and protected from the heat. Reinsert the inner liner only after the thermoplastic material has cooled down. After the MalleoLoc® is worn for half an hour, the adhesive between the plastic and the inner liner hardens due to naturally-generated body heat.

⚠ Important information

- MalleoLoc® is a prescribed product that should be applied under a physician's guidance. MalleoLoc® should only be applied in accordance with the instructions contained in these usage directions and for the listed areas of application.
- The product is intended to be fitted only once and to one patient.
- No liability is accepted for improper use of the product.
- The product should not undergo any improper modification. Failure to comply with the requirement can impair product performance and thus rule out any product liability.
- Patients with sensitive skin, known contact allergies or who are sensitive to pressure should wear a cotton sock or stocking under the MalleoLoc®.
- Should you notice any unusual changes (e. g. increased symptoms), please contact your medical provider without delay.
- A combination with other products (e. g. compression stockings) must be discussed beforehand with your medical provider.
- Avoid the use of ointments, lotions or any substances including grease or acids.
- Contact sports present a higher risk of injury. Bauerfeind does not guarantee that MalleoLoc® will prevent injuries during these activities.
- If MalleoLoc® is used during sports activities the straps should be replaced after 3 months.
- Guarantee subject to statutory regulations.
- The product can be disposed of without any restrictions.

Product care

MalleoLoc® can be washed separately by hand using a mild alkali-free detergent. To avoid damage and to preserve their long-term adhesive strength, we recommend closing the Velcro® fastenings before washing. Do not dry in direct heat (e. g. heater, sunlight, etc.).

This product has been examined by our own quality control system. However, should you have any complaint, please contact your retailer or our customer service department.

Version: July 2009

¹ Orthosis = orthopaedic appliance used to stabilize, relieve, immobilize, control or correct the limbs or the torso

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