Rev. 1 - 11/08_100504

Bauerfeind AG
Weissendorfer Strasse 5
07937 Zeulenroda-Triebes
Germany
Phone +49 (0) 36628-66-40 00
Fax +49 (0) 36628-66-44 99
E-mail info@bauerfeind-orthopaedie.de

Motion is Life: www.bauerfeind.com

MalleoTrain®

Aktivbandage
Active support
Orthèse active
Vendaje activo

Motion is Life: www.bauerfeind.com

DIN EN ISO 9001:2000
DIN EN ISO 13485:2003

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Stand der Information: November 2008

GB Dear valued customer,

Thank you for choosing a product from Bauerfeind. MalleoTrain® is a high-quality medical product.

Please carefully read through the following instructions for use. Should you have any questions please contact your doctor, medical retailer or our Customer Service department.

MalleoTrain®, an active ankle support, produces its special effect primarily during movement and therapeutic exercises. The anatomically contoured viscoelastic inserts (pressure pads) compress and massage the soft tissues of the joint. Enhanced circulation in the joint and promotion of lymph drainage lead to reduced swelling, pain relief and improved function.

MalleoTrain® must be fitted for initial use by a trained specialist, otherwise we cannot accept any liability for the product.

Indications

- Post-operative and post-traumatic irritation (e. g. after sprains)
- Joint effusion and swelling associated with degenerative changes in the ankle (osteoarthritis) and joint inflammation (arthritis)
- Muscle and tendon disorders (tendomyopathies)
- Ligamental weaknesses

Side effects

With the assumption of proper fitting, no side effects involving the body as a whole have been reported to date. Supports and orthoses¹ applied externally to the body can, if tightened excessively, lead to localized pressure or, rarely, constrict the underlying blood vessels or nerves.

Contra-indications

Hypersensitive reactions, harmful to health have not been reported to date. For patients experiencing the following conditions, such supports and orthoses should only be fitted and applied after consultation with your medical provider:

1. Skin disorders/injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.
2. Impaired sensation and circulatory disorders of the legs/feet, "e. g., Diabetes".
3. Impaired lymph drainage – including soft tissue swelling of uncertain origin, located remotely from the fitted aid.

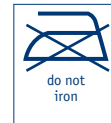
Fitting MalleoTrain®

1. Fold the upper part of the support outwards until you can grip the heel.
2. Pull the support over the foot up to the heel.
3. Fold back the folded-over part of the support.
4. Then pull it up until the bones of the ankle rest in the rounded hollows of the pads.
5. Adjust the fit of the support as necessary.

Important Information

- MalleoTrain® is a prescribed product that should be applied under a physician's guidance. MalleoTrain® should only be applied in accordance with these instructions for use and for the specified areas of application.
- The product is intended to be fitted only once, to one patient.
- No liability is accepted for improper use of the product.
- Changes must not be made to the product. Non compliance can reduce the performance of the product, thus no liability will be accepted.
- Should you notice any unusual changes (e. g., increased symptoms), please contact your medical provider without delay.
- A combination with other products (e. g., compression stockings) must be discussed beforehand with your medical provider.
- The product's unique benefits are demonstrated primarily during physical activity. The product should be removed during prolonged periods of rest (e. g., sleeping, continuous sitting).
- Avoid the use of ointments, lotions or any substances that include grease or acids.
- Guarantee is subject to statutory regulations.
- The product can be disposed of without any restrictions.

Product Care Instructions

Wash MalleoTrain® separately using hand at 30°C (86°F) with a mild detergent. Do not dry in direct heat (e. g. heater, sunlight, etc.).

This product has been examined by our own quality control system. However, should you have any complaints, please contact your retailer or our Customer Service department.

Version: November 2008

¹ Orthosis = orthopaedic appliance used to stabilize, relieve, immobilize, control or correct the limbs or the torso

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