Material

Polyamid (PA)

Polyurethan (PUR)

EVA

Polyester (PES)

Material

Polyamide (PA)

Polyurethane (PUR)

EVA

Polyester (PES)

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EN Dear customer,

thank you for choosing a product from Bauerfeind. ValguLoc® is a high-quality medical product.

Please read through these instructions for use carefully and if you have any questions contact your medical provider, medical retailer or our Technical Service department.

Deviation of the big toe toward the other toes, known medically as hallux valgus, is a consequence of splayfoot. Problems arise when the foot is placed under excessive stress resulting from high heels or shoes with an excessively tight fit around the toes. The result can be painful inflammation of the ball of the foot and the underlying synovial bursa. This is where ValguLoc® can provide relief. ValguLoc® is applied without shoes, primarily at night when the foot is not placed under load. ValguLoc® counteracts toe misalignment by constantly stretching the soft tissues and joint capsule of the foot. The big toe is gradually brought into correct alignment through the application of gentle forces.

⚠ ValguLoc® must be individually adapted to the patient by a trained orthopaedic technician in order to gain the maximum effect and optimal wear comfort from this orthosis¹.

⚠ Indications

- Bending of the big toe towards the little toe (hallux valgus) – non-surgical and surgical.

Side effects

Side effects involving the body as a whole have not been reported to date. Correct fitting is assumed. Any supports and orthoses applied externally to the body can, if tightened excessively, lead to local pressure signs or, rarely, constrict the underlying blood vessels or nerves.

¹ Orthosis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso

Contraindications

Hypersensitive reactions harmful to health have not been reported to date. In the following conditions such aids should only be fitted and applied after consultation with your medical provider:

1. Skin disorders/injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling/redness and excessive heat build-up.
2. Impaired sensation and circulatory disorders of the feet, e.g. Diabetes mellitus.
3. Impaired lymph drainage – including soft tissue swellings of uncertain origin located remotely from the fitted aid.
4. Gout.

Fitting ValguLoc®

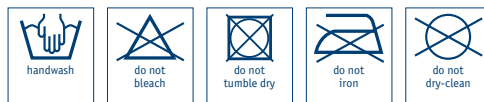
1. Undo the Velcro® fastening.
2. Push the big toe between the tape and the plastic clasp until it rests in the concave section.
3. Draw the correction tape hanging beneath the foot across the back of the foot and push through the slit in the clasp.
4. Pull the tape until the big toe appears to be correctly positioned, provided this step does not prove painful, and secure the Velcro® fastening.

Important information

- ValguLoc® is a prescribed product that should be applied under a physician's guidance. ValguLoc® should only be applied in accordance with the instructions contained in these usage directions and for the listed areas of application.
- The product is intended to be fitted only once and to one patient.
- Do not walk while wearing the product.
- It should only be used when the foot is not placed under load.
- No liability is accepted for improper use of the product.
- Improper changes must not be made for the product. Non compliance can reduce the performance of the product and thus no liability will be accepted.
- Should you notice any unusual changes (e.g. increased symptoms), please contact your medical provider without delay.

- A combination with other products (e.g. compression stockings) must be discussed beforehand with your medical provider.
- Avoid the use of ointments, lotions or any substances including grease or acids.
- Guarantee subject to statutory regulations.
- The product can be disposed of without any restrictions.

Product care and washing instructions

The support can be washed separately by hand using a mild alkali-free detergent. To avoid damage and to preserve their long-term adhesive strength, we recommend closing the Velcro® fastenings before washing. Do not dry in direct heat (e.g. heater, sunlight, etc.).

This product has been examined by our own quality control system. However, should you have any complaint, please contact your retailer or our customer service department.

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Bauerfeind France S.A.R.L.
Bauerfeind GmbH & Co.